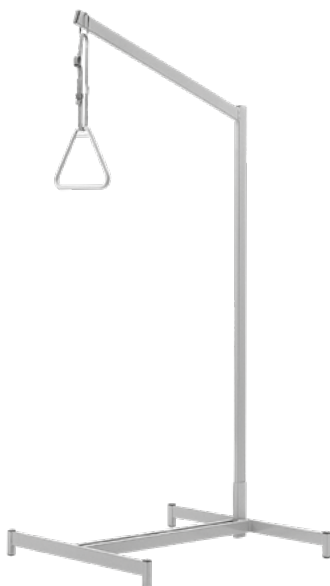
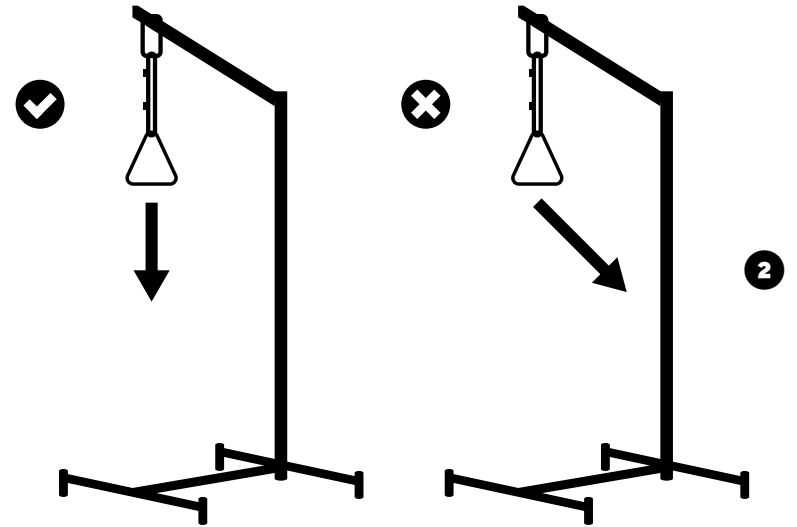
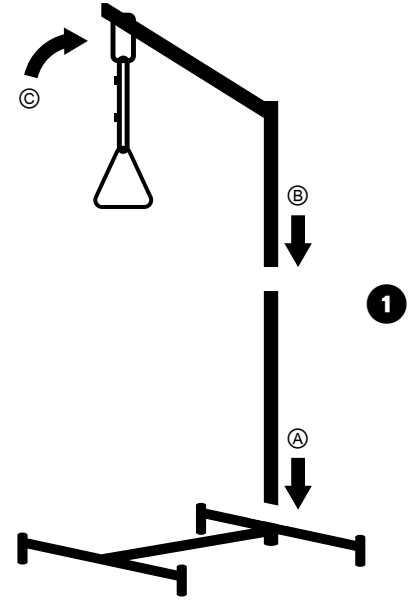
**THUASNE**

# Geneva

V014 949 020

<b>fr</b> Potence sur pied Geneva .....	4
<b>en</b> Geneva hoist frame .....	4
<b>de</b> Geneva Standaufrichter mit Fußstück .....	5
<b>nl</b> Bedgalg op voet Geneva .....	6
<b>it</b> Sollevamalati con piantana da terra Ginevra .....	7
<b>es</b> Soporte de pie Geneva .....	8
<b>sk</b> Závesná hrazda Geneva.....	9
<b>hu</b> Geneva ágykapaszkodó.....	10

## POTENCE SUR PIED GENEVA

### Description/Destination

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## GENEVA HOIST FRAME

### Description/purpose

The Geneva hoist frame is designed for adults and children who are confined to bed or a chair to help them change position. It can be used with most standard beds and low chairs.

### Main components

- Structure: epoxy-treated steel
- Handle: polypropylene
- Strap: polypropylene

### Indications

Designed to help the patient raise the upper body when lying on a bed or seated in a chair.

### Contraindications

Do not use in the event of known allergy to any of the components.

### Precautions

#### Important warnings

Do not leave or store the product outside.

Do not overload the hoist. Maximum capacity: 80 kg (approximately two thirds of the body weight).

Always place the frame in a stable position on a flat, solid surface.

Avoid excessive lateral force. When using the hoist with an adjustable-height bed or chair, keep the hoist frame far enough away from moving parts.

Do not position the hoist frame beside furniture, but behind the bed head or chair back.

Follow the advice of the healthcare professional who prescribed or supplied the product and consult them in case of doubt.

It is recommended that a healthcare professional assembles and adjusts the device and supervises the first use.

Before using this product, check that it is stable and rigid and that all the components are properly attached.

Verify the product's integrity before every use.

Before each use, check that the strap seams and material and the fixings of the triangular handle are intact and solidly attached.

Do not use the product if it is damaged.

Do not use when transferring the patient from a bed to a chair or wheelchair, for example.

Do not use the product for any purpose other than its intended use: risk of injury or damage to equipment.

Beware of the risks of pinching when handling the device.

Monitor the use of the product by people with limited physical capacity or people with cognitive impairments.

Never allow children to play with the product.

Do not make any mechanical changes to the product, as this could compromise its safety. Only original spare parts may be used.

### Undesirable side-effects

Risk of discomfort in the event of incorrect adjustment of the device.

Any serious incidents related to the product should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is resident.

### Instructions for use

#### Assembly

1. Unpack the four parts.
2. Place the foot on the floor. **1**
3. Insert the straight square tube into the socket on the foot. **A**
4. Insert the angled tube into the straight square tube so that the arm is above the foot. **B**
5. Attach the triangular handle by inserting the metal pin into one of the five holes in the hoist. **C**
6. Adjust the handle height to suit the patient (see Adjustment).
7. Position the hoist behind the bed or chair, making sure the arm bearing the handle is centered above the patient and the patient does not apply any lateral force to the arm when changing position by holding on to the handle. **2**

### Adjustment

The height and position of the handle are adjustable. The strap can be lengthened or shortened and the pin of the metal ring can be moved to a different hole in the hoist so that the patient can grasp the handle with both hands.

The Geneva hoist frame is easy to dismantle, transport and store.

### Maintenance and care

#### Maintenance

If any part of the product becomes visibly damaged, contact a specialized dealer or healthcare professional.

The strap and the fixings of the triangular handle must be replaced every five years.

If necessary, replacement parts and original accessories are available from your Thuasne product dealer.

#### Care

Clean with a mild cleaning agent and a soft cloth.

If necessary, use a mild contact disinfectant or 70° alcohol. In an institution, follow the recommendations of the hygiene manager.

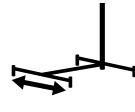
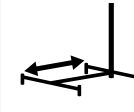
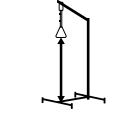
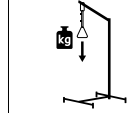
#### Storage

Store at room temperature, in a dry, ventilated place.

#### Disposal

Dispose of in accordance with local regulations.

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©Thuasne - 2034301 (2022-07)