THUASNE

# Pedipro Plus

<b>fr</b>	<b>Talonnettes silicone avec inserts anatomiques</b> .....	5
<b>en</b>	<b>Silicone heelpieces with anatomical inserts</b> .....	7
<b>de</b>	<b>Ferseneinlagen aus Silikon mit anatomischen Pelotten</b> .....	8
<b>nl</b>	<b>Siliconen zolen met anatomische inserts</b> .....	10
<b>it</b>	<b>Alzaticchi in silicone con inserti anatomici</b> .....	12
<b>es</b>	<b>Taloneras de silicona con insertos anatómicos</b> .....	14
<b>pt</b>	<b>Palmilhas de calcanhar de silicone com inserções anatómicas</b> .....	16
<b>da</b>	<b>Indlægshæle i silikone MEDanatomiske indsatse</b> .....	18
<b>fi</b>	<b>Anatomisilla tyynyillä varustetut silikonikantapäätuet</b> .....	20
<b>sv</b>	<b>Silikoninlägg med anatomiska delar</b> .....	22
<b>el</b>	<b>Υποπτέρνια σιλικόνης με ανατομικά ενθέματα</b> .....	23
<b>cs</b>	<b>Silikonové podpatěnky s anatomicky tvarovanými vloženými prvky</b> .....	26
<b>pl</b>	<b>Podpiętki silikonowe z wkładkami anatomicznymi</b> .....	27
<b>lv</b>	<b>Silikona papēži ar anatomiskiem ieliktniem</b> .....	29
<b>lt</b>	<b>Silikoninės pakulnės su anatominiais įdėklais</b> .....	31
<b>et</b>	<b>Silikonist kannapadjad anatoomiliste vahetükkidega</b> .....	33
<b>sl</b>	<b>Silikonski petni nastavki z anatomskimi vložki</b> .....	35
<b>sk</b>	<b>Silikonové podpätienky s anatomickými výstelkami</b> .....	37
<b>hu</b>	<b>Szilikon sarokemelő anatómiai betétekkel</b> .....	39
<b>bg</b>	<b>Силиконови подложки за пета с анатомични вложки</b> .....	41
<b>ro</b>	<b>Talonete din silicon cu inserții anatomice</b> .....	43
<b>ru</b>	<b>Силиконовые подпяточники с анатомическими вставками</b> .....	45
<b>hr</b>	<b>Silikonski jastučići za petu s anatomskim umetcima</b> .....	47
<b>zh</b>	<b>帶有符合解剖形狀插片的硅胶足跟护垫</b> .....	49
<b>ar</b>	<b>وسائد كعب سيليكونية مزودة بحشوات متوافقة مع شكل الكعب</b> .....	53

		Europe	UK
	1	36 - 39	3 <sup>1/2</sup> - 5 <sup>1/2</sup>
	2	40 - 46	6 <sup>1/2</sup> - 11

<b>fr</b>	Pointure - Europe - UK
<b>en</b>	Shoe size - Europe - UK
<b>de</b>	Schuhgröße - Europa - UK
<b>nl</b>	Schoenmaat - Europa - VK
<b>it</b>	Numero di scarpa - Europa - Regno Unito
<b>es</b>	Número de pie - Europa - Reino Unido
<b>pt</b>	Número do sapato - Europa - Reino Unido
<b>da</b>	Størrelse - Europa - UK
<b>fi</b>	Kengän koko - Eurooppa - Yhdistynyt kuningaskunta
<b>sv</b>	Skostorlek - Europa - Storbritannien
<b>el</b>	Νούμερο υποδημάτων - Ευρώπη - ΗΒ
<b>cs</b>	Velikost obuvi - Evropa - Spojené království
<b>pl</b>	Numer buta - Europa - Wielka Brytania
<b>lv</b>	Apavu izmērs - Eiropa - Apvienotā Karaliste
<b>lt</b>	Avalynės dydis - Europa - JK
<b>et</b>	Suurus - Euroopa - Ühendkuningriigid
<b>sl</b>	Velikost obutve - Evropa - UK
<b>sk</b>	Veľkosť - Európa - Spojené kráľovstvo
<b>hu</b>	Cipőméret - Európa - UK (Egyesült Királyság)
<b>bg</b>	Размер - Европа - Обединено кралство
<b>ro</b>	Mărimea pantofului - Europa - Regatul Unit
<b>ru</b>	Размер обуви - Европа - Великобритания
<b>hr</b>	Broj cipela - Europa - UK
<b>zh</b>	鞋码-欧洲-英国
<b>ar</b>	المقاس - أوروبا - المملكة المتحدة

Décharge	Restaure l'équilibre	Correction biomécanique
Off-loading	Balance restoration	Biomechanical correction
Entlastung	Stellt das Gleichgewicht wieder her	Biomechanische Korrektur
Losmaken	Herstelt het evenwicht	Biomechanische correctie
Scarico	Restaure l'equilibrio	Correzione biomeccanica
Descarga	Restaure el equilibrio	Corrección biomecánica
Alívio do peso	Restaure o equilíbrio	Correção biomecânica
Aflastning	Genopretter ligevægten	Biomekanisk korrektion
Vaputus	Palauttaa tasapainon	Biomekaaninen korjaus
Avlastning	Förbättrar balansen	Biomekanisk korrigering
Αποφόρτιση	Αποκαθιστά την ισορροπία	Βιο-μηχανικός έλεγχος
Snížení zátěže	Obnovuje rovnováhu	Biomechanická korekce
Odciążenie	Produkt przywraca równowagę	Korekcja biomechaniczna
Atbrīvošana	Līdzsvara atjaunošana	Biomehāniskā korekcija
Apkrovis sumažinimas	Pusiausvyros palaikymas	Mechaninė biologinė korekcija
Koormuse vähendamine	Taastab tasakaalutunnetuse	Biomehaaniline korrektsioon
Razbremenitev	Vzpostavi nazaj ravnovesje	Popravek biomehanske motnje
Uvoľnenie	Obnovenie rovnováhy	Biomechanická korekcia
Tehermentesítés	Az egyensúly helyreállítása	Biomechanikus korrekció
Отбремявяване	Възстановява баланса	Биомеханична корекция
Descărcare	Restabilirea echilibrului	Corecție biomecanică
Разгрузка	Восстановление равновесия	Биомеханическая коррекция
Rasterečenje	Uspostavljanje ravnoteže	Biomehānička korekcija
减压	恢复平衡	生物力学矫正
تخفيف ثقل	إعادة التوازن.	تصحيح ميكانيكي حيوي

fr

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---

en

## SILICONE HEELPIECES WITH ANATOMICAL INSERTS

### Description/Destination

- 1 Silicon provides even distribution of pressure and shock absorption.
- 2 Blue, flexible, anatomically-shaped inserts in the area of the heel spur.
- 3 Components promoting hygienic freshness.
- 4 Unique self-adhesive system helps them to stay in place in the shoe.

### Composition

Silicone - polyamide - silver.

### Properties/Mode of action

- Off-loading.
- Balance restoration.
- Biomechanical correction.

### Indications

- Heel and foot pain (metatarsalgia, plantar fasciitis).
- Rheumatoid foot.
- Relief for tendinitis.

### Contraindications

- Do not apply the product in direct contact with broken skin or an open wound without an adequate dressing.
- Do not use in the event of known allergy to any of the components.

### Precautions

- In the event of discomfort, significant hindrance, pain, variation in limb volume, abnormal sensations or change in colour of the extremities, remove the device and consult a healthcare professional.
- Choose the appropriate size to fit, referring to the size chart.
- For hygiene and performance reasons, do not re-use the product for another patient.
- Verify the product's integrity before every use.
- Do not use the device if it is damaged.
- Do not bring the product into direct contact with greasy substances (ointment, cream, etc.).

### **Undesirable side-effects**

This device can cause skin reactions (redness, itching, irritations...). Any serious incidents occurring related to the device should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is resident.

### **Instructions for use/Application**

Position the self-adhesive tab under the silicone surface of the heel insert (self-adhesive surface against the heel insert).

Clean the interior of the shoe.

Remove the protective paper from the adhesive tab.

Place the heelpiece flat in the shoe, pressing the rim of the product against the heel of the shoe.

N.B.: this product is designed for a specific side, indicated by "RIGHT" for your right shoe and "LEFT" for your left shoe.

Press down hard with your hand to make sure the tab is properly stuck to the shoe.

Repeat this operation with the other heelpiece.

THUASNE has supplied you with extra tabs so that you can remove the heel inserts and use in different shoes.

### **Care**

It is recommended to wash the product every day, or after each use. Hand wash. Do not dry clean. Do not use detergents, fabric softeners or aggressive products (products containing chlorine). Do not tumble-dry. Do not iron. Dry away from any direct heat source (radiator, sun, etc.). Dry flat.

### **Storage**

Store at room temperature, preferably in the original packaging.

### **Disposal**

Dispose of in accordance with local regulations.

This medical device is a regulated health product with CE marking

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©Thuasne - 2030902 (2022-07)
**THUASNE UK Ltd**
Thuasne SAS – SIREN/RCS Nanterre 542 091 186